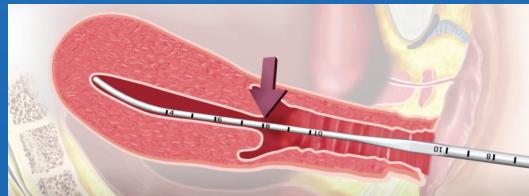
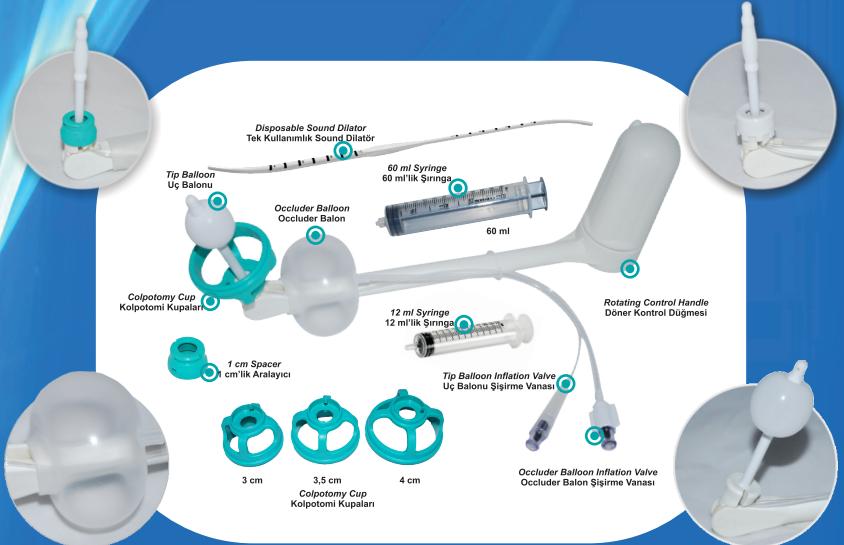
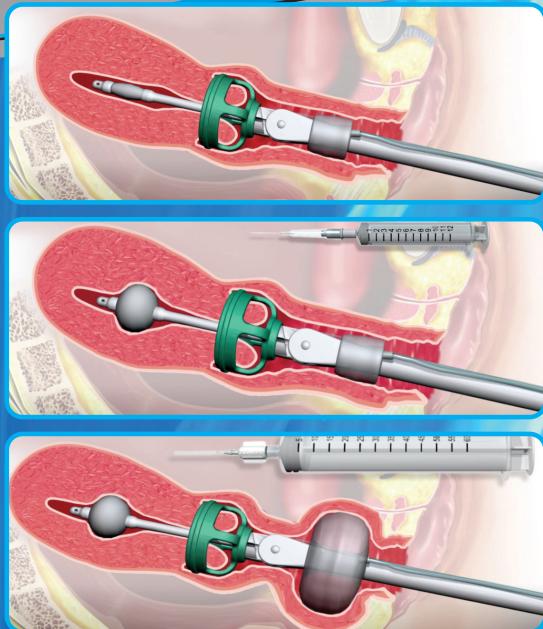
**prestij**  
**MEDICAL**  
[www.prestijmedical.com](http://www.prestijmedical.com)  
Gsm: 0532 297 00 25 / Tel: 0212 531 31 75



ClearView®

Clinical Innovations®

## ClearView® TOTAL Laparoscopic Hysterectomy Uterine Manipulator


## ClearView® Uterine Manipulator Laparoscopic
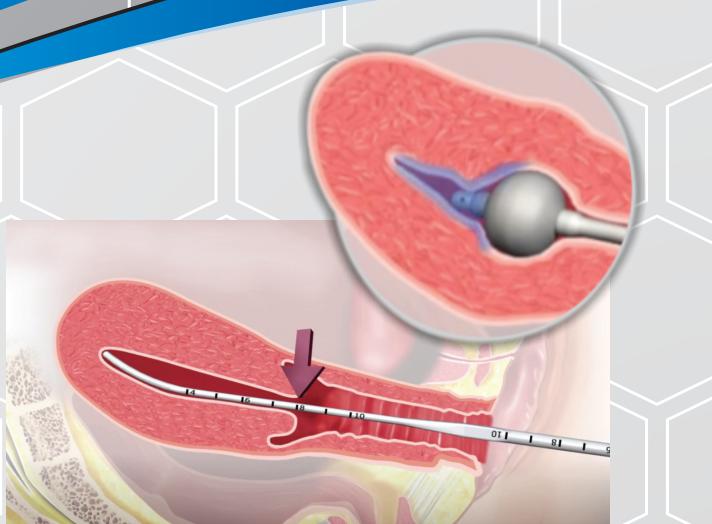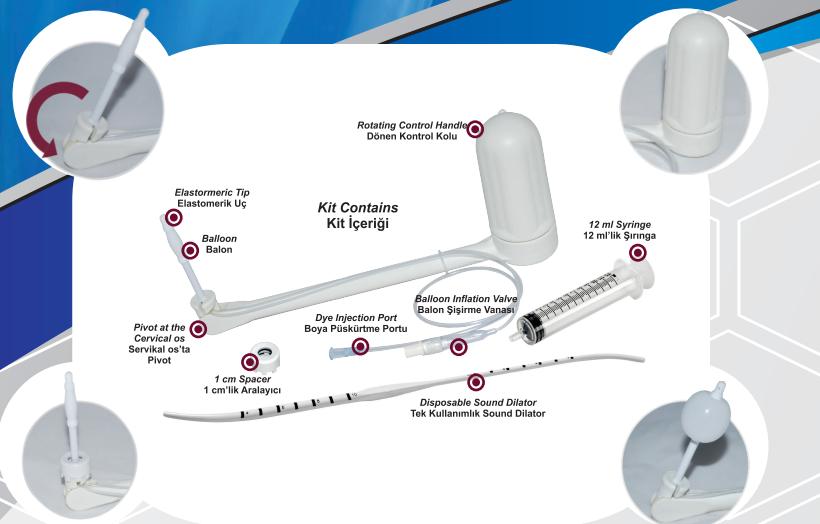
# Reference Guide for Laparoscopic Hysterectomy

## ClearView Total Uterine Manipulator
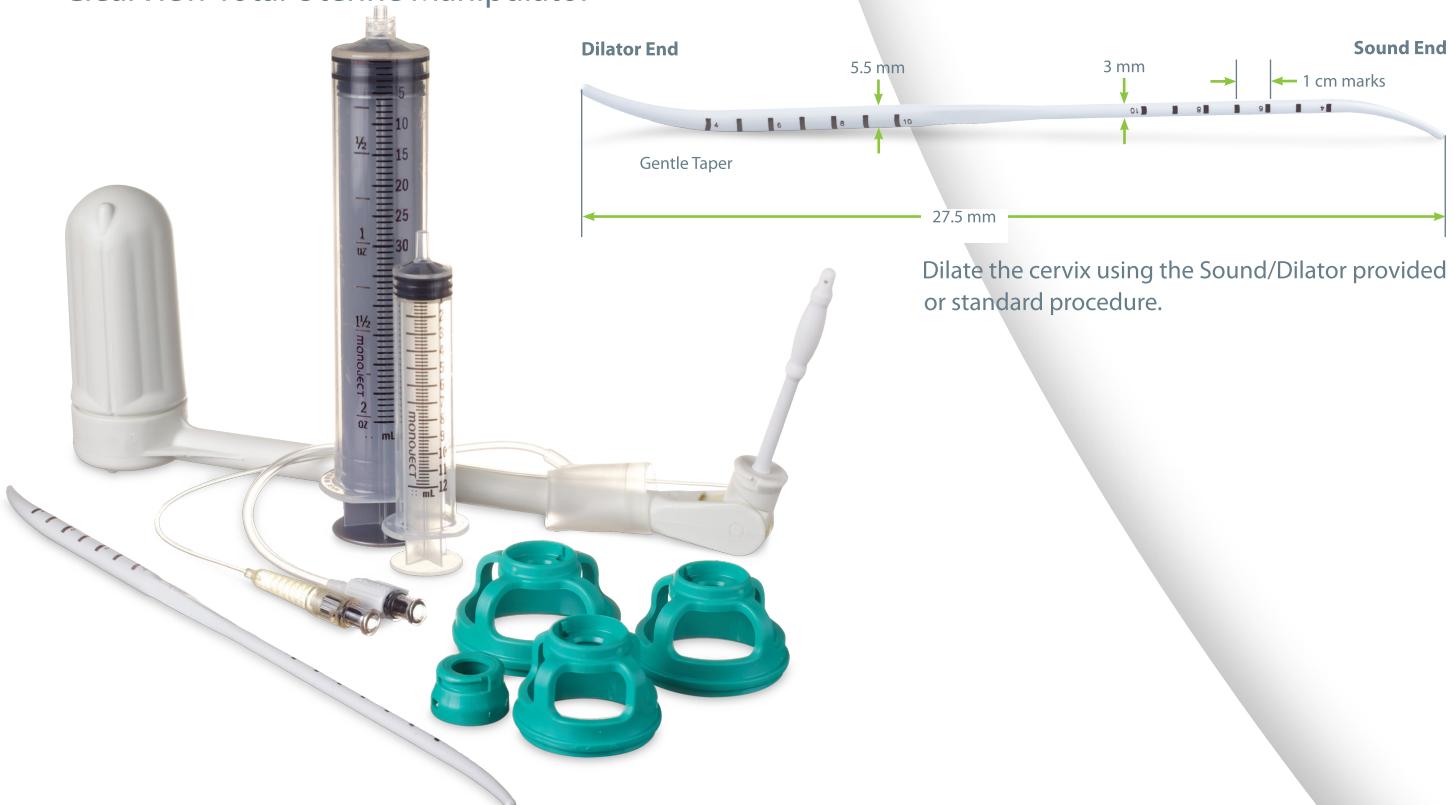
# Dilate Cervix

- UM750: If uterus sounds to 7 to 8 cm, attach 1 cm spacer.  
If uterus sounds to 8 to 9 cm, no spacer is needed.
- UM950: If uterus sounds to 9 to 10 cm, attach 1 cm spacer.  
If uterus sounds to >10 cm, no spacer is needed.


## Manipulate Uterus as Needed
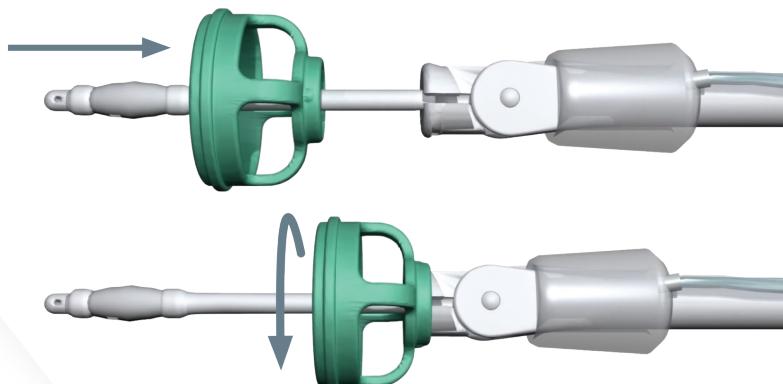
## Choose Cup Size and Attach Cup

Choose the correct size of colpotomy cup from the sterile tray to properly fit the patient.

Carefully place the colpotomy cup over the deflated ClearView Total tip until it touches the tip support or spacer (if used).

Twist the colpotomy cup clockwise until it snaps tightly in place.

## After Completion of Procedure

Deflate intrauterine balloon and vaginal occluder balloon with syringes and slowly remove the device from the vagina.

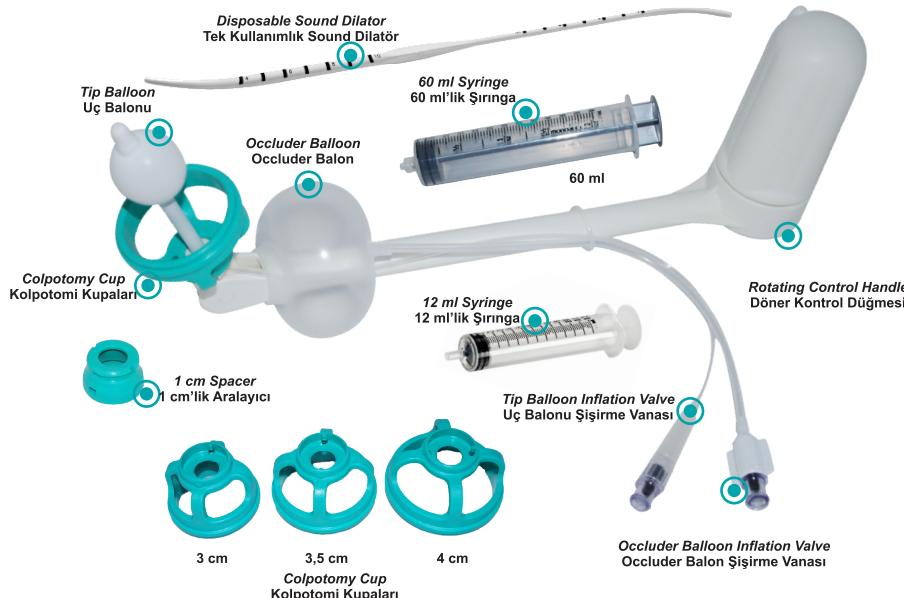
Check to ensure that the device is intact and that no parts remain in the patient.

Discard the device.

# ClearView® TOTAL

Laparoscopic  
Hysterectomy  
Uterine Manipulator

## Optimal visualization. Complete uterine control.

ClearView TOTAL is a single-use, sterile device that allows for uterine manipulation and clearly defines where to make the colpotomy incision. In addition to the features of the traditional ClearView, the ClearView TOTAL includes a pre-attached occluder balloon as well as 3 different sizes of colpotomy cups, conveniently packaged in a sterile inner tray which can be removed and placed aside until use. The colpotomy cups were designed for improved visibility during insertion and are colored for easy identification when making the colpotomy incision. ClearView TOTAL is designed specifically for laparoscopic procedures where it is desirable to locate the position of the vaginal fornices, typically during Total Laparoscopic Hysterectomy.

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## ClearView® TOTAL Benefits

### Colpotomy Cups Compatible With Any Scalpel

Each ClearView TOTAL includes 3 different sizes (3,3.5,4cm) of colpotomy cups to ensure the proper size is available. The colpotomy cups are made from a proprietary material which, unlike many other colpotomy cups, can be used with both electrosurgical and harmonic scalpels. The colpotomy cups were designed for improved visibility during insertion and are colored for easy identification when making the colpotomy incision.

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Each ClearView TOTAL includes 3 different sizes (3,3.5,4cm) of colpotomy cups to ensure the proper size is available. The cups, which are made from a proprietary material, can be used with both electrosurgical and harmonic scalpels.

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#### ► Maintain Pneumoperitoneum / Pnömoperitonumu Muhabaza Eder

The pre-attached occluder balloon requires no assembly. It serves as a seal to prevent the loss of the pneumoperitoneum once the colpotomy incision has been made.

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### Occluder Balloon

The pre-attached occluder balloon requires no assembly. It serves as a seal, preventing the loss of the pneumoperitoneum once the colpotomy incision has been made. It is filled with 60-120cc of sterile liquid using the included 60mL syringes.

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### Adjustable Tip Length

A 1cm spacer is included in the ClearView TOTAL kit. The spacer adjusts the tip length, effectively changing a 9cm or 7cm tip into an 8cm or 6cm tip respectively. If the spacer is needed to optimize sizing it simply snaps on in seconds.

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### Optimal Patient Position

The rotating control handle facilitates one-handed control, enabling the surgeon to manipulate the uterus through the drape without breaking the sterile barrier. The reference "bump" on the handle helps the surgeon know which direction the tip is positioned. The design also enables the patient to remain in the supine position.

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## ClearView TOTAL Laparoscopic hysterectomy

### WARNING

- Do not use the ClearView Total if it has been opened or damaged.
- Do not reuse. Discard after one procedure. Function may be impaired through reuse or cleaning.
- The device is difficult to clean after exposure to biological materials and may cause adverse patient reactions if reused. **Clinical Innovations will not be responsible for any direct, incidental, or consequential damages resulting from the reuse of this device.**
- As with all uterine manipulators, a careful clinical examination should be performed prior to use. Certain clinical conditions may present a uterus which is more prone to perforation or bleeding.
- Sound the uterus for depth and direction prior to use. **Do not** use the ClearView Total as a sound.
- Do not use a tip which is longer than the sounded depth. Apply a spacer, or use a shorter-tipped device, so insertion is 1 cm or less the sounded depth.
- Do not use the device without first testing the tip and Occluder balloons.
- The tip balloon is designed to hold up to 10 cc of inflation.
- Do not inflate balloons with air.**
- Use of an incorrect sized ColpoCup could result in patient injury.
- Do not force the tip of the device into a cervical os that is insufficiently dilated.
- A non-inflated, or under-inflated, tip balloon may result in the expulsion of the device from the uterus during use.
- Do not attempt extensive uterine manipulation without a clear laparoscopic view.
- Do not employ extreme or excessive force when manipulating.
- Do not attempt to remove the device by pulling on any exposed tubing. Remove by pulling on the handle only.
- Do not allow prolonged contact between cutting instrument and ColpoCup.

### INDICATIONS

The Clearview Total is intended for use in laparoscopic procedures where it is desirable to delineate the vaginal fornices, and the surgeon intends to remove or access intraperitoneal tissue through the vagina by use of a colpotomy or culdotomy incision; such as laparoscopically assisted vaginal hysterectomies or total laparoscopic hysterectomies while maintaining pneumoperitoneum by sealing the vagina while a colpotomy is performed.

### CONTRAINdicATIONS

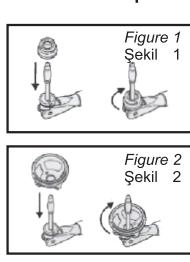
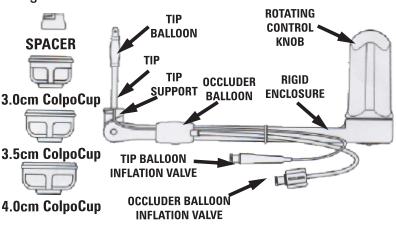
Pregnancy or suspicion of pregnancy, known or suspected uterine or tubal infection, an IUD in place, uterine anomaly especially anomalies with a small cavity, in cases in which the surgeon deems it inadvisable or find it difficult to insert.

### PRECAUTION

Confirm that the ColpoCup and/or spacer is locked into place prior to starting the procedure.

### INSTRUCTIONS FOR USE

1. Check the sterile tray for integrity. Remove the device. Using the small syringe (included in package), inflate the tip balloon with sterile saline (10 cc maximum). Remove the syringe and make sure the balloon stays inflated. Reattach the syringe and remove the fluid, deflating the balloon, prior to patient insertion. Using the large syringe included in the package, test the integrity of the Occluder balloon by inflating with 30-60 cc of sterile saline (do not use air); the deflate prior to patient insertion.
2. Sound the uterus using the combination Sound/Dilator provided.  
For height 7 cm ClearView device; if the uterus sounds to less than 8 cm attach a spacer.  
For height 9 cm ClearView device; if the uterus sounds to less than 10 cm, attach a spacer.  
To attach a spacer, with tip balloon deflated, position spacer with the larger-open end facing towards the pivot and carefully slide the spacer over the tip until it touches the tip support; then twist the spacer clockwise until it snaps tightly in place (see Figure 1). The manipulator tip should be positioned 1 cm below the maximum sounded depth of the uterus.
3. Choose the correct size of ColpoCup to properly fit the patient and then carefully place the ColpoCup over the deflated ClearView Total tip until it touches the tip support or spacer (if used).  
Twist the ColpoCup clockwise until it is in **snaps tightly in place** (see Figure 2).
4. If indicated, dilate the cervix using Standard procedures, or use the combination Sound/Dilator provided.
5. Lubricate the distal manipulator tip and balloon. After placing a speculum, slowly insert the tip into the uterine cavity. Completely insert the tip into the uterus until the bottom of the ColpoCup abuts the external cervix. Confirm that the distal end of the ColpoCup is secured in the vaginal fornices.
6. Using the small syringe, slowly inflate the tip balloon with sterile saline until the base of the ColpoCup is pulled tight against the cervix (10 cc maximum). Continue to hold the plunger of the syringe as it is removed from the inflation valve, to prevent reflux of fluid back into the syringe. Do not over inflate the tip balloon.
7. Gently pull on the manipulator to test for proper engagement of the cuff in the uterus. Remove the speculum and rotate the manipulator clockwise to the anteverted position with the manipulator handle between the patient's legs.
8. The uterus can now be manipulated by rotating the ClearView Total's handle. Clockwise rotation elevates the uterus to the anteverted position; counterclockwise rotation retroverts the uterus. The uterus can also be moved laterally by directing the handle to one side or the other.
9. Just prior to making the colpotomy incisions, inflate the Occluder balloon with 60-120 cc of sterile saline (do not use air).
10. After completion of the procedure, deflate the Occluder balloon and remove the device, in its entirety, from the vagina. **Check to ensure that the device is intact and that no parts remain in the patient.**

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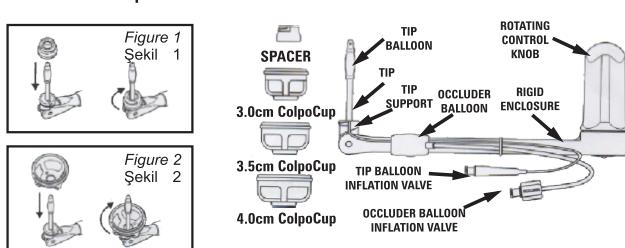
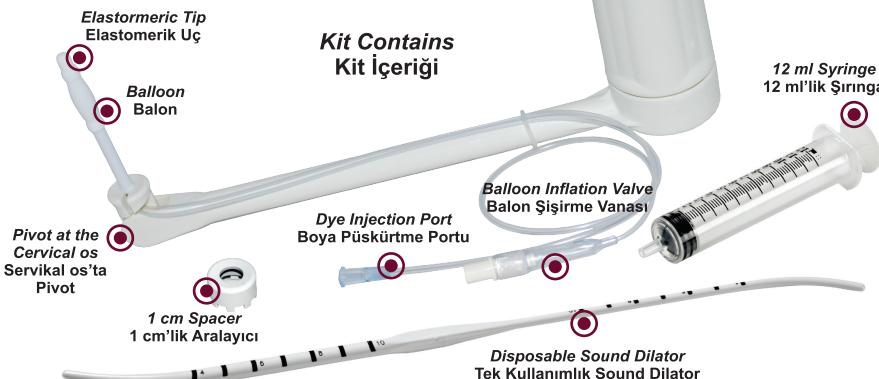
### Optimal visualization.

### Complete uterine control.

ClearView is an innovative, completely disposable, uterine manipulator. It provides surgeons with optimal visibility and complete uterine control during laparoscopic procedures. With its pivot point at the cervical os, ClearView provides 120° anteversion, 90° retroversion, as well as lateral movement; for the greatest motion range of any uterine manipulator. No tenaculum is required for insertion; and with the patient in the preferred supine position; ClearView's light weight, balanced-design is self-holding, eliminating the need for an assistant.

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### Complete ClearView Kit

A state-of-the-art uterine manipulator designed for complete uterine control, convenience, and safety. ClearView is an innovative, completely disposable, uterine manipulator. It provides surgeons with optimal visibility and complete uterine control during laparoscopic procedures. ClearView provides 120° degrees anteversion and 90 degrees retroversion, as well as lateral movement; for the greatest motion range of any uterine manipulator.

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### Recent Device Upgrades

You may or may not be aware, but ClearView recently went through a few upgrades. We listened to the feedback our customers provided and adjusted the device accordingly. The ClearView is now more reliable than ever!

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### Pivot at the Cervical os & Greatest Range of Motion

Having the pivot at the cervical os increases mobility and visibility. ClearView provides a full 210° of articulation for unmatched exposure.

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### Rotating Control Handle

The rotating control handle facilitates one-handed control, enabling the surgeon to manipulate the uterus without breaking the sterile barrier. The reference "bump" on the handle helps the surgeon know which direction the tip is positioned.

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### Why choose ClearView®? Neden ClearView®?

#### Greatest Range of Motion / Neden ClearView?

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#### Optimal Patient Position / Optimal hasta pozisyonu

Maximum manipulation with the patient in the preferred supine position as opposed to the lithotomy position required by other devices.

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The rotating handle provides one-handed surgeon control w/o breaking the sterile barrier or needing an assistant.

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### Self-holding Device

The light-weight, balanced, design, allows the ClearView to be moved into position and left, freeing the hands of the surgeon and eliminating the need for an assistant.

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### Adjustable Tip Length

A 1cm spacer is included in the ClearView kit. The spacer adjusts the tip length, effectively changing a 9cm or 7cm tip into an 8cm or 6cm tip respectively. If the spacer is needed to optimize sizing it simply snaps on in seconds.

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### Effective Chromotubation

During surgery, typically laparoscopy, the status of the tubes can be inspected as a dye is injected, in a process known as chromotubation, into the uterus and shown to pass through the tubes. Laparoscopic chromotubation has been described as the gold standard of tubal evaluation. The ClearView is an effective device to perform such tests.

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[www.prestijmedical.com](http://www.prestijmedical.com)

### WARNINGS

- + Sound uterus for depth and direction before use. **NEVER use the ClearView® as a sound.**
- + Never use a tip which is longer than the sounded depth. Apply spacer so depth of insertion is 1 cm less than sound depth.
- + Never attempt extensive uterine manipulation without a clear laparoscopic view of the uterus.
- + Never use the device without testing cuff first.
- + Do not employ extreme or excessive force when manipulating.
- + Never force the tip into a cervical os that is insufficiently dilated.
- + As with all uterine manipulators, a careful clinical examination should be performed prior to use. Certain clinical conditions may present a uterus which is more prone to perforation and bleeding.
- + A non-inflated or under-inflated cuff may result in the expulsion of the device from the uterus during use.
- + Dye injection should be performed slowly. Rapid injection may cause uterine damage or result in fallopian tube spasm.
- + If preferential flow is noted on chromotubation, occlude the side with free flow to assess contralateral flow.

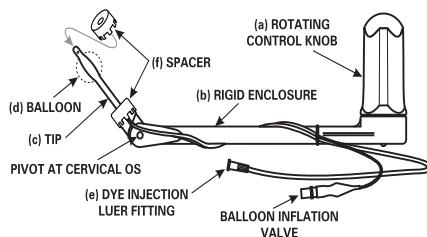
### ADVERSE REACTIONS

- + Cramping
- + Infection
- + Perforation of uterine wall
- + Uterine spasm with accompanying temporary physiologic blockage of patent fallopian tubes

### Instructions for Use

1. Check the sterile tray for integrity. Remove the instrument. Using the included syringe, inflate cuff (balloon at distal tip) with fluid (10cc maximum). Remove the syringe and make sure that the cuff stays inflated. Reattach the syringe and remove the fluid, thus deflating the balloon.
2. Sound the uterus using the combination Sound-Dilator provided. For the **7cm ClearView®** device; if the uterus sounds to less than 8 cm, attach a spacer. For the **9cm ClearView®** device; if the uterus sounds to less than 10 cm, attach a spacer. To attach a spacer, position spacer with the snap fingers towards the pivot and carefully slide the spacer over the tip (with balloon deflated) until it **snaps in place** at the base of the tip (see diagram). The uterine manipulator tip with its injection ports should be positioned 1 cm below the maximum depth of the uterus by sounding. Dilate the cervix, if indicated, using standard procedures or utilize the combination Sound-Dilator provided.
3. Lubricate the distal manipulator tip and cuff. After placing a speculum, slowly insert the tip into the uterine cavity. Completely insert the tip into the uterus until the face of the cervical stop abuts the external cervix.
4. Slowly inflate the cuff with fluid until the cervical stop is pulled tight against the cervix (10cc max.). Hold the plunger of the syringe as it is removed from the reflux valve to prevent the reflux of fluid back into the syringe. Do not over inflate the cuff.
5. Gently pull on the manipulator to test for proper engagement of the cuff in the uterus. Remove the speculum, rotate the manipulator to the antevert position, with the manipulator handle between the patient's legs.
6. Manipulate the uterus by rotating the handle. Clockwise rotation elevates the uterus to the antevert position. Counterclockwise rotation retroverts the uterus. The uterus can also be moved laterally by directing the handle to one side or the other.
7. If desired, inject media through the dye injection port to test patency of the fallopian tubes.
8. When the procedure is complete, insert the syringe into the reflux valve and draw off the fluid from the cuff. Move the uterus to its natural position and remove from the uterus and vagina.

**Check to ensure that no parts remain in the patient.**

The Clinical Innovations ClearView® is both a uterine manipulator and a uterine injector designed for single use for complete uterine control by the attending laparoscopist. It is a sterile product made of molded plastic material which meets USP/Class VI recommendations for implant testing. This product is designed as a single unit body with (a) a rotation control knob which controls tip, (b) an enclosure, (c) a dual lumen obturating tip which pivots at the cervical os, (d) a soft inflatable uterine cavity balloon with external inflation tubing for securing the device in the uterine cavity, (e) a luer fitting to accommodate a syringe for contrast media injection through the distal tip assembly, and (f) a spacer for reducing insertion depth by 1 cm. Extended length is 32.5 cm (12.8 inches). Outer diameter of distal tip is 5.1 mm (0.20 inches).

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© Copyright 2013
© Copyright 2013 ABD Mali P/N 050-0709 Rev. A